AM-HV Oil
ARTSMedia High Viscosity Oil

High Viscosity Mineral Oil (Paraffin Oil)
Aceite mineral de alta viscosidad
(aceite de parafina)

Product No.
N.º de producto506050
506100**Technical Support**
Soporte técnico support@artsmedia.dk +45 5350 4530 www.artsmediadenmark.comARTSMedia Denmark
Kongevejen 149,
DK-2830 Virum, Denmark**Indications for use**

AM-HV Oil is a high viscosity mineral oil (d=0.86-0.88g/ml) pre-washed with ultra pure water for use in In Vitro Fertilization (IVF), Intracytoplasmic Sperm Injection (ICSI) and assisted reproductive technologies (ART).

The oil permits small volumes of medium to be used for fertilization and embryo culture / handling. The oil overlay prevents media evaporation, which would result in increased osmolarity, and to avoid fluctuations in the pH and the temperature.

For professional use only.

Device Description

50- and 100-ml glass bottles, single packed, provided sterile. The AM-HV Oil is a pharmaceutical grade light mineral oil*, pre-washed and sterile filtered light paraffin oil.

* Liquid paraffin oil is a mineral oil and is a by-product of crude oil distillation.

Quality Control Specifications

Density: 0.86-0.88g/ml
Viscosity: 95-140cP at 30°C
Endotoxin:< 0.1EU/ml USP <85>
Peroxide Value (POV): < 0.1mEq/kg
Sterility: No growth
(Ph. Eur. 2.6.1, USP<71>)
1-Cell Mouse Embryo Assay:
≥ 80% blastocysts embryos developed to expanded blastocyst at 120 hours.

The results of each batch are stated on a Certificate of Analysis, which is available at www.artsmediadenmark.com.

Storage

Keep away from sunlight at 15-25°C in original container. Do not freeze. The products can be used up to 1 month after opening, when aseptic conditions are maintained and the products are stored at 15-25°C.

Stable after transport (max. 5 days) at elevated temperature (< 37°C.)

Warnings

- Do not use after expiry date.
- Do not use if the product is cloudy or shows any evidence of microbial contamination.
- Do not use if seal of the container is damaged or open upon delivery.
- Do not re-sterilize after opening.
- Discard in accordance with local regulations for disposal of medical devices.

Precautions

- All human organic samples should be handled as if capable of transmitting HIV or hepatitis.
- Oil use should be conducted using aseptic techniques.
- Always wear protective clothing when handling samples.
- The oil may appear cloudy if removed from the incubator and allowed to stand at room temperature for extended periods of time. Such cloudiness usually disappears if the oil is re-equilibrated by replacing it in the incubator for several hours. If it does not disappear discard the product.

Instruction for use

AM-HV Oil is pre-washed and ready to use. The oil should be pre-warmed prior to use.

To maintain pH, temperature, and osmolality, make sure that the medium is completely covered with oil.

The oil should be used according to the established procedures and protocols optimized for the individual clinic.

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