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501010

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ARTSMedia Denmark  
Kongevej 149,  
DK-2830 Virum, Denmark

**Indications for use**

AM-IVC Medium is a culture medium intended for in vitro fertilization and in vitro culture of human gametes and embryos from fertilization until the blastocyst stage of development (day 5). The medium can also be used for embryo transfer.

**Device Description**

10 ml glass bottles, individually packed, provided sterile. AM-IVC Medium is a bicarbonate-buffered balanced salt solution, with Physiological Salt, Amino Acids, Calcium Lactate Pentahydrate, EDTA, Gentamicin Sulphate 10 µg/ml, Glucose, L-glutamine, Human Serum Albumin (HSA), Insulin, Sodium Hyaluronate, Sodium Pyruvate, Sodium Bicarbonate, and Phenol Red.

**For professional use only.**

**Quality Control Specifications**

pH: 7.2 – 7.45 (Ph Eur. 2.2.3, USP <791>)  
Osmolality: 257-275 mOsm/kg  
(Ph Eur. 2.2.35, USP <785>)

Appearance: Clear and particulate free  
Sterility: No growth  
(Ph Eur. 2.6.1, USP <71>)

Endotoxin: < 0.05 EU/ml (USP <85>)  
Mouse Embryo Assay (MEA) one cell system: ≥ 80 % embryos developed to expanded blastocysts at 96 hours.

A certificate of Analysis, for each batch is available at [www.artsmmediadenmark.com](http://www.artsmmediadenmark.com)

**Storage**

Store in original container between 2-8 °C and avoid exposure to sunlight. Do not freeze. Stable after transport (max. 14 days) at elevated temperature (≤ 37 °C).

**Warnings**

- Do not use after expiry date
- Do not use if the product is cloudy or shows any evidence of microbial contamination
- Do not use if seal of the container is damaged or open upon delivery
- Do not use medium that shows any evidence of particulate matter or has changed color
- Do not re-sterilize after opening
- Do not reuse
- Single-use only. Discard excess (unused) media that has been

warmed in accordance with local regulations for disposal of medical devices

- AM-IVC Medium contains gentamicin sulfate. Appropriate precautions should be taken to ensure that the patient is not sensitized to this antibiotic (allergic reaction).

**Precautions**

- Caution: U.S. Federal law restricts this device to sale by or on the order of a physician
- Media use should be conducted using aseptic techniques
- Always wear protective clothing when handling samples
- This product contains albumin, a derivative of human blood. All blood products should be treated as potentially infectious. Source material used for this product was tested and found negative for HBsAg, Anti-HIV-1/HIV2, HCV, HCV-RNA, and HIV-RNA. Furthermore, source material has been tested for parvovirus B19 and found to be non-elevated.

**General notes before use**

AM-IVC Medium is bicarbonate buffered and should pre-equilibrate in a CO<sub>2</sub> incubator set at 37 °C and 5-6 % CO<sub>2</sub>. Note: Adjust incubator settings to a higher CO<sub>2</sub> concentration if the altitude is above sea level.

To maintain pH, temperature, and osmolality, make sure that the medium is completely covered with oil.

**Instruction for use**

AM-IVC Medium should be used according to the established procedures and protocols optimized for the individual clinic.

AM-IVC Medium must equilibrate for a minimum of 2 hours in 5-6 % CO<sub>2</sub> at 37 °C prior to use. Equilibration can also be done overnight.

**Fertilization:** Oocytes and prepared sperm can be handled/incubated in AM-IVC Medium in preparation of, or during fertilization by IVF/ ICSI. The medium does not contain HEPES. Thus, to maintain a physiological pH, CO<sub>2</sub> incubation prior to use is necessary.

**Embryo culture:** AM-IVC Medium is a medium for continuous one step embryo culture without

media change. It does not need to be renewed or exchanged during culture. Embryos can be cultured from day 1 to blastocyst stage (shorter culture times are possible).

**Example:** Prepare dishes with 40-60 µl droplets of AM-IVC Medium with an oil overlay. Equilibrate a CO<sub>2</sub> incubator at 37 °C and 5-6 % CO<sub>2</sub> overnight or for a minimum of 2 hours prior to use. Per droplet, 1-4 COCs can be inseminated by incubation with sperm or denudated oocytes injected (ICSI). Culture embryos in freshly CO<sub>2</sub> equilibrated medium drops.

**Embryo transfer:** The embryos are prepared and transferred to the uterus in 20-30 µl freshly CO<sub>2</sub> equilibrated AM-IVC Medium. Flush the transfer catheter with AM-IVC Medium prior to use.

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